

Product Information Sheet

Product Name:

Hemoclean®

Company:

KRD CO., LTD. (Korea)

Intended Use:

- Hemoclean is just applying to the Hemodialysis machines which were allowed to use Peracetic acid disinfectant and the hemodialysis machine could adjust the dilution ratio and cleaning
- The dilution ratio and cleaning period should be followed machine manual.
- Do the residual tests of Peracetic acid with Peracetic Acid Potency Test Kit after machine disinfect and confirm the result is less than 0.5 ppm. Bellow table is about the recommended dilution ratio and cleaning period from each hemodialysis machine manual. As time as the sterilize mode finished, do the residual test to confirm the chemical residual is not remained in the machine.



Disinfectant residual tests could be followed machine manual.

Main Features/ USPs:

- cleaner and disinfectant for hemodialysis machine
- User's safety
- Speedy and powerful sterilizing effect
- Excellent cleaning effect
- Convenience for users
- Environmentally friendly

Distributor Price:

Contact us for pricing details:

Jiyoung YANG - jyyang@krdkorea.com, Tel: 82-2-890-7876,

Cell : 82-10-9960-1749, Fax. : 82-890-7868